



ZOLL EMV+ Ventilator

PM Due Alarm

ZOLL
an Asahi Kasei company

ZOLL EMV+ Ventilator - PM Due Alarm

- The PM Due Alarm will occur at startup when the last annual preventative maintenance is greater than 365 days.
- This is a low-priority message to help remind the user that an annual maintenance and calibration is due.
- The ZOLL EMV+ ventilator is still operational and can be used on patients.
- When appropriate, the ventilator should be sent for preventative maintenance.



To disable the PM Due Alarm, to allow the EMV+ to be used to treat a patient, follow these steps:

Step 1

- Turn the ZOLL EMV+ on.

Note: After the EMV+ is turned on, the PM Due alarm will be displayed.




To disable the PM Due Alarm, to allow the EMV+ to be used to treat patient, follow these steps:

Step 2

- Press Red "X" button to silence the alarm and to clear the display.


Note: After the Red "X" button has been pressed, the Start Menu will be displayed, allowing you to select a patient size.

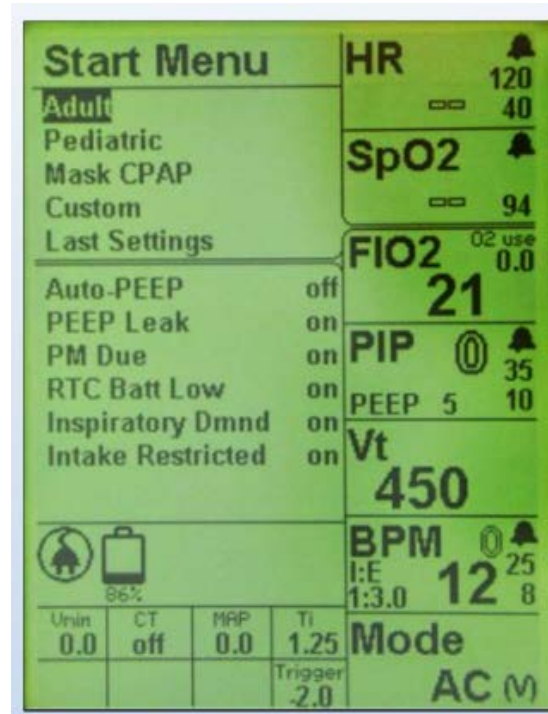

To disable the PM Due Alarm, to allow the EMV+ to be used to treat a patient, follow these steps:

Step 3

- Select Patient Adult or Peds
- Press Green "✓" ACCEPT
 - For Adult Patients

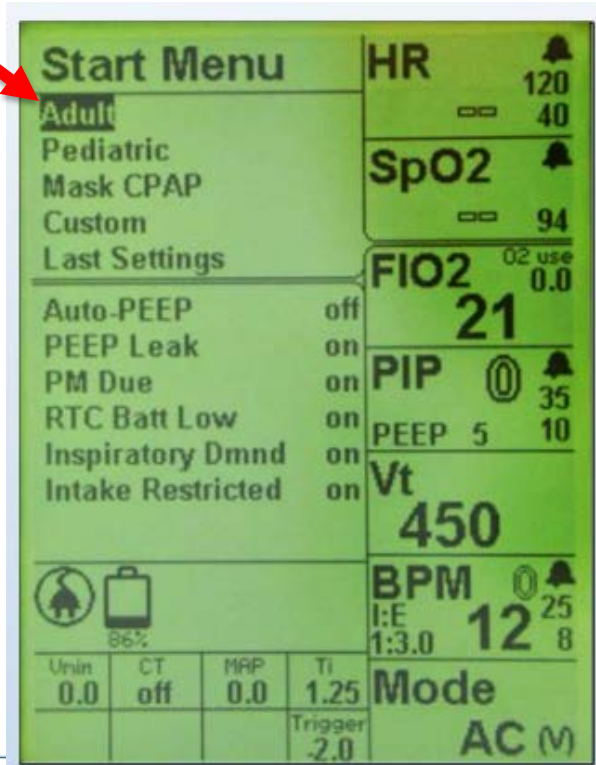


To disable the PM Due Alarm, to allow the EMV+ to be used to treat patient, follow these steps:

Step 4

- Press Menu Button

Note: After the menu button is pressed the display will show the Menu items and the Alarm Config menu item will be selected.

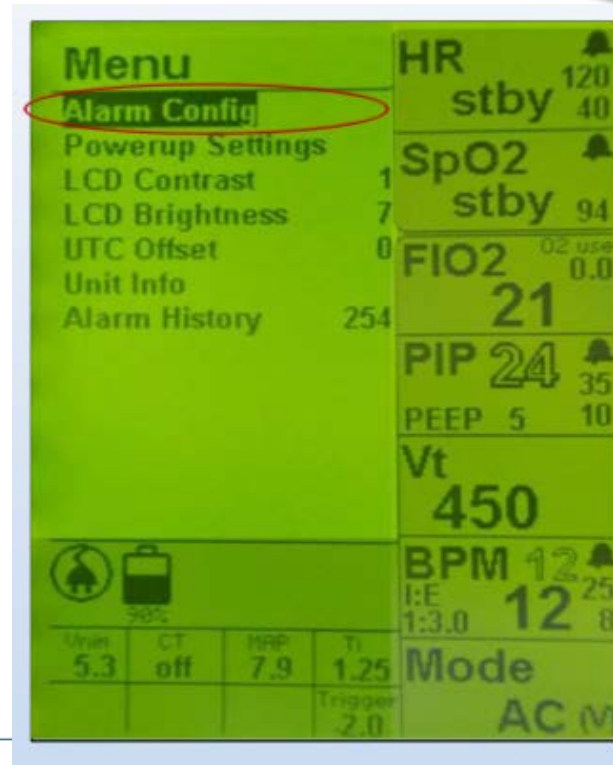


To disable the PM Due Alarm, to allow the EMV+ to be used to treat patient, follow these steps:

Step 5

- Press Green "✓" ACCEPT


Note: After the "Accept" button is pressed the display will show the Alarm Config items.

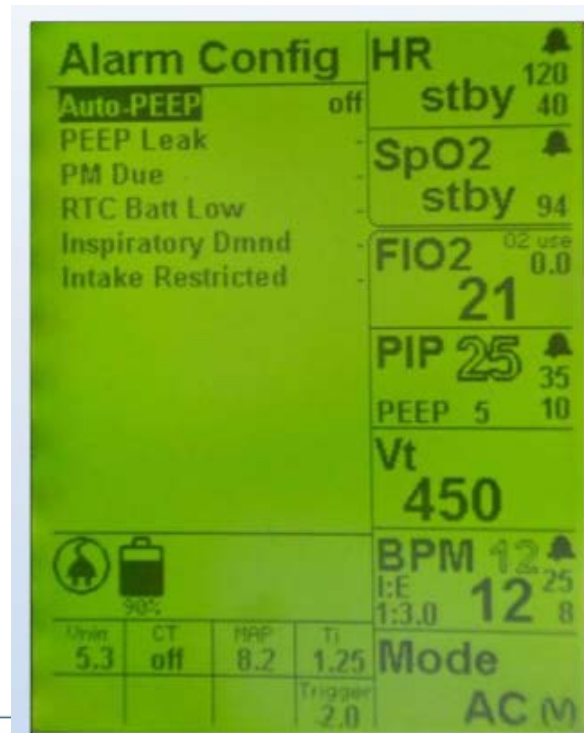

To disable the PM Due Alarm, to allow the EMV+ to be used to treat patient, follow these steps:

Step 6

- Turn Selector dial to select the PM Due menu item

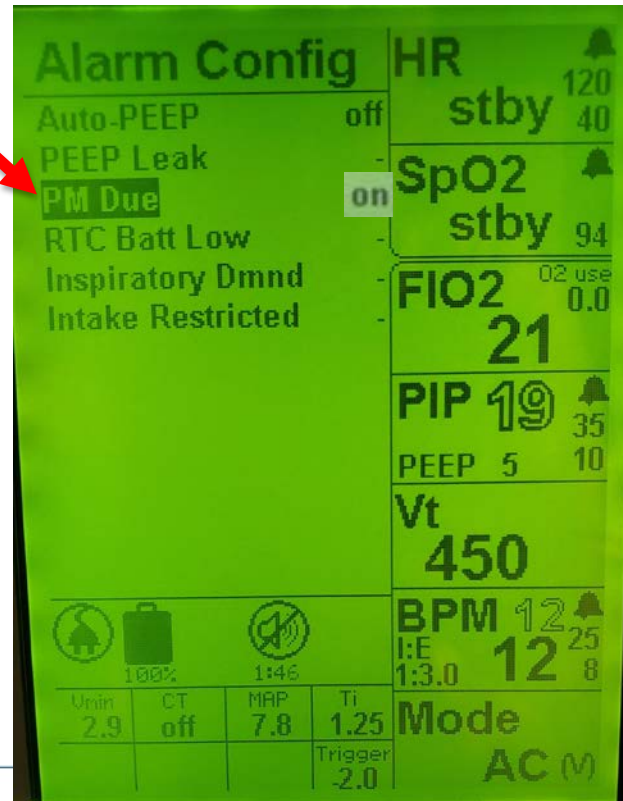


To disable the PM Due Alarm, to allow the EMV+ to be used to treat patient, follow these steps:

Step 7

- Press Green "✓" ACCEPT


Note: After the "Accept" button is pressed the PM Due "ON" will be selected, allowing you to change the alarm mode.

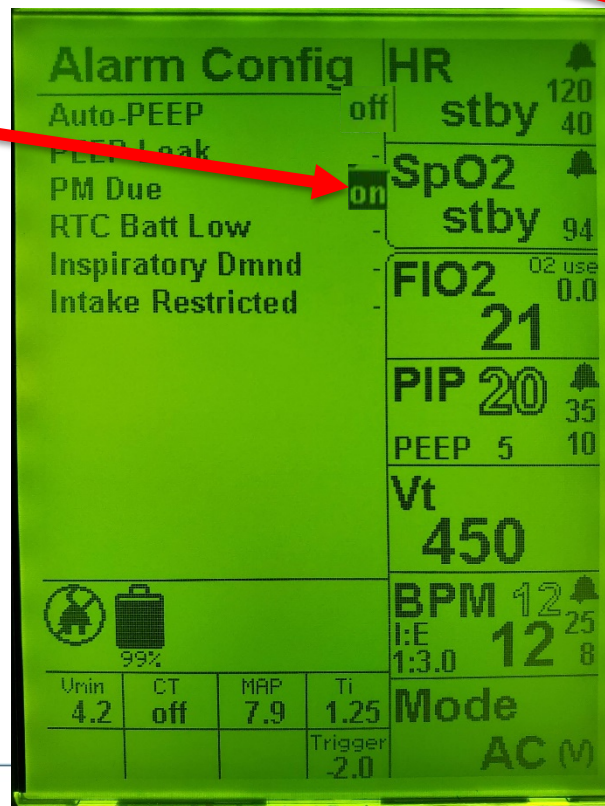

To disable the PM Due Alarm, to allow the EMV+ to be used to treat patient, follow these steps:

Step 8

- Turn Selector dial so that the PM Due "off" is displayed.

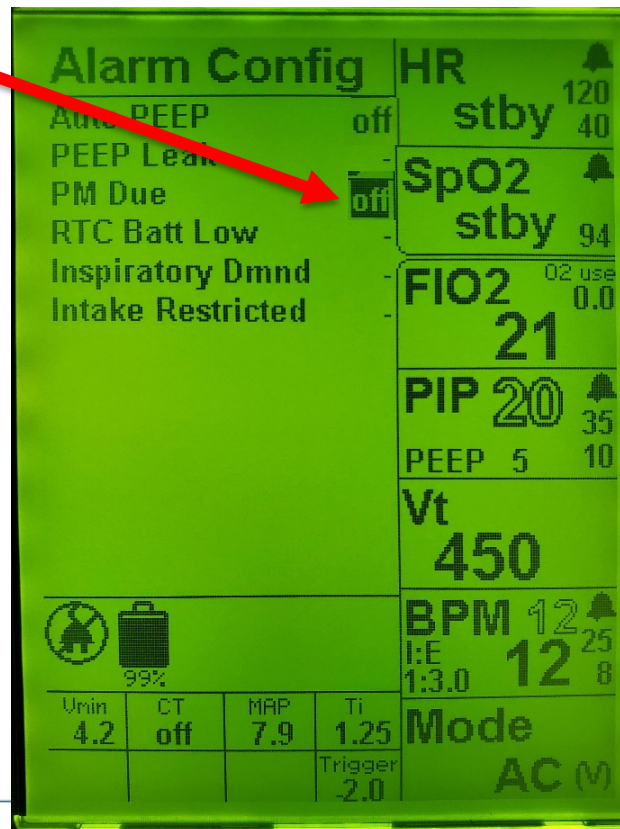

To disable the PM Due Alarm, to allow the EMV+ to be used to treat patient, follow these steps:

Step 9

- Press Green "✓" ACCEPT

Note: This will turn off the PM Due alarm for this patient use.

Alarm Config				HR	
Auto-PEEP	off			stby	120
PEEP Leak	off				40
PM Due	off			SpO2	
RTC Batt Low				stby	94
Inspiratory Dmnd				FIO2	02 use
Intake Restricted				21	0.0
				PIP	19
				PEEP	5
				Vt	450
				BPM	12
				I:E	1:3.0
				Mode	AC (M)
				Trigger	2.0



To disable the PM Due Alarm, to allow the EMV+ to be used to treat patient, follow these steps:

Step 10

- Press Red "X" button 3 times to exit all menus.

Note: This will allow you to adjust ventilator settings to treat the patient.

The PM Due alarm will activate again on a power ON cycle. The above steps will need to be performed to disable the PM Due alarm for the next patient.





THANK YOU

Please contact ZOLL Technical Support at milsupport@zoll.com
for any additional questions or support.

© 2021 ZOLL Medical Corporation
All rights reserved.